


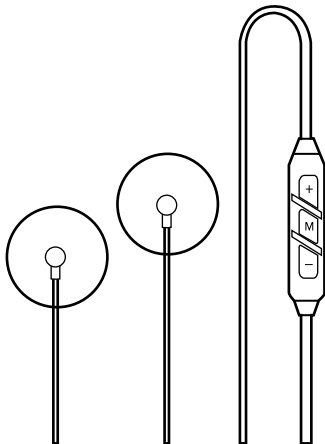

TENS MASSAGER

LLM333



Relax
Muscles



Quick
Effect



Relieve
Pain



Protection
Carry Case



2 Reusable
Gel Pads




8
Intensity



No Batteries or
Charging Required





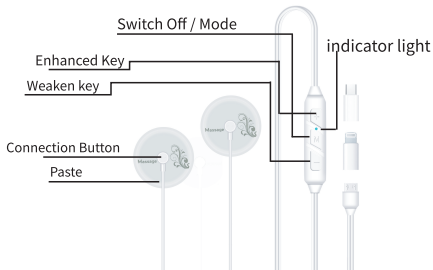


Dear Customer,
Congratulations on your Lifelong Purchase. Our name stands for high-quality and thoroughly tested products. Please read these instructions carefully before use.

SPECIFICATIONS



1. Connection buckle connection patch.
2. The patch should be attached to the skin at the same time.
3. Plug in a cell phone or power supply.
4. Press **M** button to power on (Some mobile phones need to open the OTG function) .
5. Long press the **M** button to power off.

This product is a portable line-controlled massage device, insert mobile phone or power supply.

1. 3 buttons:  to start/shutdown  for strengthening or weakening.
2. After plugging in the indicator light will be on and then off, which means the power supply has been connected then, long press button  to start and the indicator light will stay on. The indicator light is blue.
3. Press the button , there are 6 modes, cycle, hammer, scraping, kneading, fiery pot, massage. The device will be automatically shut down after 30 minutes of operation. The timer will moves to Switch mode. After switching mode, the intensity level is restored to zero.
4. 9 strength levels,  for strengthening (level 0-9)  for weakening (level 9-0).
5. The device will be automatically turned off after 30 minutes without strength. The strength will return to zero after unloading (leaving the skin) for 1 second.
6. Long press button  for shutdown and the indicator light will be off.

REMINDER

If the cell phone light is not on, please hit it manually.
OTG function (settings -- more settings -- OTG -- open).
If the plug does not work, you can use the switch head.

PRODUCT ICON

- A patient who is undergoing treatment and has abnormal physical condition.
- Pregnant women with malignant tumors and pregnant women with abnormal heart, blood pressure and brain nerve.
- Patients who are taking medication and whose body temperature exceeds 38 degrees celsius (during fever).
- Patients with cutaneous perception disorders or dermal abnormalities in patients with infectious diseases.
- Patients with acute illness requiring recuperation have abnormal bleeding.
- Patients who cannot freely express their wishes.
- High peripheral circulation disorder due to diabetes mellitus.
- Patients with perceptual impairment who have skin allergies to electrodes.
- Do not use this product on the following parts, otherwise it may cause accidents or skin and body problems:
 1. Places where plastic surgery was performed on the chest and pericardium metal and plastic implants above neck, head and face.
 2. Mucosal areas such as wounds, eyes, mouth and pudenda, spine tattoos.
 3. The abdomen at physiological stage where hair removal was done

5. Users of life-sustaining medical electronic instruments such as artificial heart and lung.
6. Users of wearable medical electronic instruments such as electrocardiogram scanners.

THIS PRODUCT IS A NON-MEDICAL DEVICE.

Consumer Warranty Card

Dear Customer,

Thank you for choosing a Lifelong consumer product. All Lifelong Consumer products are designed and manufactured to the highest standards to deliver high quality performance, as well as easy installation and use. At Lifelong, we believe in providing not only service, but adding value to your purchase. The warranty has therefore been designed especially for you with your interest at heart.

Warranty Service

All Lifelong consumer products are covered against manufacturing defects from them date of purchase.

Name of the product: _____

Model: _____

Warranty coverage: 1 year

Please note: Purchase receipt is necessary for warranty verification.

Customer Care: customercare@lifelongindia.com

Customer Details

Name: _____

Address: _____

Home Number: _____

Office Number: _____

E-mail Address: _____

Product Details

Model No.: _____

Serial No.: _____

Purchase Date: _____

Invoice Number: _____

Online Site: _____

Please log onto to www.lifelongindiaonline.com and complete the online warranty form with your personal and product details within 14 days.

Terms and Conditions:

1. This warranty is void if:
 - a. The completely filled warranty card is not presented at the time of servicing the product.
 - b. The product is not operated according to the instructions given under the user manual.
 - c. Damages are caused by lightning, abnormal voltage, water or other liquid intrusion, fire, flood, accident, negligence or improper handling.
 - d. Product has been damaged due to installation, repairs, alterations or modifications by unauthorized service organizations or persons.
 - e. Product label specifying the model number, serial number and production code has been removed and altered.
 - f. Defects or parts requiring replacement due to ordinary wear and tear, corrosion, rust or stains, scratches, dents on the casing or paintwork of the product.
 - g. Claims damaged and/or missing parts (accessories) after 7 days from the original date of goods received.
 - h. Defects or faults in product which have been used for commercial/industrial purposes or which have been rented/leased or have been otherwise subject to non-household/non-domestic use.
2. Repairs or replacements will be carried out by authorized service provider.
3. During the limited warranty period, Lifelong or its authorized service provider will repair without charge the defective unit inclusive of labor and parts and restore the unit to its optimum working condition. All defective parts used for the warranty repair should be surrendered to Lifelong and/or its authorised servicer.
4. All expenses incurred in collecting the unit (s) or part(s) thereof from authorised service provider as well as any other expenses and incidentals will be borne by the consumer.
5. Lifelong obligations are limited to the repair and replacement of the defective product. Except as set forth above, there are no other express or implied warranty and all warranted, conditions or other terms implied by statute or common law (including any warranty of satisfactory quality, merchantability or fitness for a particular purpose) are excluded to the fullest extent permitted by the law.
6. Lifelong total liability for damages relating to or arising out of the purchase or use of the product regardless of the type or cause of such damage of the form of characterization of the claim asserted (e.g. contract or tort) shall not exceed the original purchase price paid for the product.
7. However in no event shall Lifelong and Lifelong authorized distributors be liable for any punitive, special incidental, indirect or consequential losses or damages whatsoever (including without limitation, damages for the loss of revenue, business, profits, goodwill, or contracts, business interruptions, loss of business information or any other pecuniary loss.) Whether or not Lifelong has been advised of the possibility of such damages. These limitations shall apply not with standing the failure of the essential purpose of any limited warranty. This limited warranty does not affect consumer's statutory rights under the law.
8. No carrier, dealer or employee is thereof authorized to make modifications to this warranty and you should not reply on any such representation. Lifelong reserves the right to amend the terms and conditions if necessary.

Manufactured for & Marketed by :
Lifelong Online Retail Private Limited

Made in China

For queries and complaints:
please contact: customer@lifelongindia.com
www.lifelongindiaonline.com